



# DESIFOR- QUICK PLUS *wipes*

## Liquid quick disinfectant agent

### Area of application

Ready to use, alcohol-based quick disinfectant agent for residue-free and quick disinfection of all areas of medical facilities and in the food industry. According to MPG: Aldehyde-free disinfection and cleaning of alcohol-proof, non-invasive medical products. According to biocidal products decree: For disinfection and cleaning of alcohol-proof medical inventories as well as surfaces of all kinds. Due to its perfume and residue-free formulation, it is particularly suitable for the food industry, kitchens and kitchen areas in medical facilities.

### Application

Completely dampen the cleaned area to be disinfected with DESIFOR QUICK PLUS and allow to dry. Observe exposure time

### Technical information

Carton of 2 x 1 l bottles,  
2 spray heads

order No. 00523

### Note

Do not use on alcohol-sensitive surfaces (e.g. acrylic glass). Apply max. 50 ml/m<sup>2</sup>. Total applied quantity per room must not be greater than 100 ml/m<sup>2</sup> room floor space.

### Active ingredients

100 g solution contains 45.0 g ethanol.

Identification pursuant to EC Regulation 1272/2008 (CLP): Caution; GHS02

Product code GD 0

Model reg. no. N-22859, N-22860

CE 0482

Apply biocidal products with care.  
Always read label and product information before use.

| Area of application/efficiency   | Concentration/<br>exposure time | Note   |
|--|---------------------------------|--|
| Medical area: VAH-listed, without mechanics, increased organic pollution, bactericidal and yeasticidal | conc. 5 minutes                 | effective within 1 min., according to expertise                |
| Food industry, public facilities: Bactericidal and yeasticidal (EN 1276, EN 1650, EN 13697)            | conc. 1 minutes                 | 20 °C high and low contamination                               |
| Tuberculosis-bacteria (M. terrae)  | conc. 30 sec.                   | pursuant to EN 14348, low contamination                        |
| Virucidal (incl. HIV, HBV, HCV)  | conc. 30 sec.                   | pursuant to RKI recommendation Federal Health Bulletin 01-2004 |
| Noroviruses (MNV)  | conc. 1 minutes                 | pursuant to EN 14476, low contamination                        |
| Rotaviruses  | conc. 30 sec.                   | pursuant to EN 14476   |

### Product characteristics

- VAH-listed; listed in IHO list of disinfectants and virucidal agents
- Free from fragrance and colourings (according to LFGB)
- Dries residue-free
- Aldehyde-free quick disinfectant
- Bactericidal (incl. MRSA), yeasticidal, tuberculocidal
- Effective against Norovirus and Rotavirus
- Limited virucidal effect – all coated viruses e.g. HBV/HIV, HCV, Vaccinia, BVDV, Influenza
- HACCP Marketing Certificate is available
- Pursuant to EU Directive 93/42/EWG for medical products

All information contained in our brochures is accurate to the best of our knowledge and experience. No liability can be derived from the information herein. For details about the product, please refer to the safety data sheet. (GB) 0318

pH conc **0 2 4 8 10 12 14**

**DR.SCHNELL GmbH & Co. KGaA**  
Taunusstraße 19 · 80807 Munich  
Tel. +49 89 3506080  
info@dr-schnell.com  
www.dr-schnell.com

*Wir leben Hygiene*  
**DR.SCHNELL**  
QUALITÄT UND SERVICE