



DESIFOR- QUICK PLUS

Liquid quick disinfectant agent

Area of application

Ready to use, alcohol-based quick disinfectant agent for residue-free and quick disinfection of all areas of medical facilities and in the food industry. According to MPG: Aldehyde-free disinfection and cleaning of alcohol-proof, non-invasive medical products. According to biocidal products decree: For disinfection and cleaning of alcohol-proof medical inventories as well as surfaces of all kinds. Due to its perfume and residue-free formulation, it is particularly suitable for the food industry, kitchens and kitchen areas in medical facilities.

Application

Completely dampen the cleaned area to be disinfected with DESIFOR QUICK PLUS and allow to dry. Observe exposure time

Technical information

Carton of 2 x 1 l bottles,
2 spray heads

order No. 00523

Note

Do not use on alcohol-sensitive surfaces (e.g. acrylic glass). Apply max. 50 ml/m². Total applied quantity per room must not be greater than 100 ml/m² room floor space.

Active ingredients

100 g solution contains 45.0 g ethanol.

Identification pursuant to EC Regulation 1272/2008 (CLP): Caution; GHS02

Product code GD 0

Model reg. no. N-22859, N-22860

CE 0482

Apply biocidal products with care.
Always read label and product information before use.

Area of application/efficiency	Concentration/ exposure time	Note
Medical area: VAH-listed, without mechanics, increased organic pollution, bactericidal and yeasticidal	conc. 5 minutes	effective within 1 min., according to expertise
Food industry, public facilities: Bactericidal and yeasticidal (EN 1276, EN 1650, EN 13697)	conc. 1 minutes	20 °C high and low contamination
Tuberculosis-bacteria (M. terrae)	conc. 30 sec.	pursuant to EN 14348, low contamination
Virucidal (incl. HIV, HBV, HCV)	conc. 30 sec.	pursuant to RKI recommendation Federal Health Bulletin 01-2004
Noroviruses (MNV)	conc. 1 minutes	pursuant to EN 14476, low contamination
Rotaviruses	conc. 30 sec.	pursuant to EN 14476

Product characteristics

- VAH-listed; listed in IHO list of disinfectants and virucidal agents
- Free from fragrance and colourings (according to. LFGB)
- Dries residue-free
- Aldehyde-free quick disinfectant
- Bactericidal (incl. MRSA), yeasticidal, tuberculocidal
- Effective against Norovirus and Rotavirus
- Limited virucidal effect – all coated viruses e.g. HBV/HIV, HCV, Vaccinia, BVDV, Influenza
- HACCP Marketing Certificate is available
- Pursuant to EU Directive 93/42/EWG for medical products

All information contained in our brochures is accurate to the best of our knowledge and experience. No liability can be derived from the information herein. For details about the product, please refer to the safety data sheet. (GB) 0717

pH conc **0 2 4 8 10 12 14**

DR.SCHNELL GmbH & Co. KGaA
Taunusstraße 19 · 80807 Munich
Tel. +49 89 3506080
info@dr-schnell.com
www.dr-schnell.com

Wir leben Hygiene
DR.SCHNELL
QUALITÄT UND SERVICE